

NOWONDER™

NOWONDER™ Nasal Cleanser intended for use whenever exposed to harmful elements such as bacteria, viruses, pollen and pollution.

NOWONDER™ Nasal Cleanser has been clinically tested and shown to be safe for human use.

Who can use NOWONDER™ Nasal Cleanser?

NOWONDER™ sprayer is suitable for adults and children older than 12 years of age.

WARNINGS

Intended for single person use only.

Do not use the spray if you are sensitive to any of its components. Stop use if you feel significant burning or irritation.

If the spray gets into your eyes, rinse with water.

Do not swallow.

How to store NOWONDER™ Nasal Cleanser?

Store at room temperature.

Do not freeze.

Discard the sprayer on the "Best Used By" date printed on the sprayer or 6 months after first opening, whichever comes first.

How to use NOWONDER™ Nasal Cleanser??

NOWONDER™ sprayer may be applied 3 to 4 times daily (2 sprays per nostril each time) or up to 6 times daily, as needed.

NOWONDER™ sprayer contains 0.9 FL OZ (25mL) of solution.

Once applied it becomes effective immediately.

Can be used with other nasal products.

INGREDIENTS

Purified water (Aqua), Sodium Chloride, Citric Acid, Hypromellose, Sodium Nitrite, Benzalkonium Chloride

Saline Based

Non-habit forming

Won't cause drowsiness

Steroid free

Non-GMO Project Verified




Manufacturer: SaNOtize R&D Corp.

Distributor: NoWonder Inc.
Lewes, Delaware, 19958 USA


DIRECTIONS



Step 1 - Blow your nose gently before using the spray.

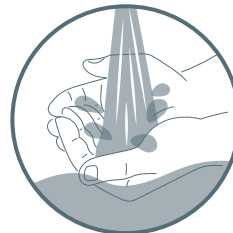


Step 2 - Wash your hands thoroughly with soap and water.

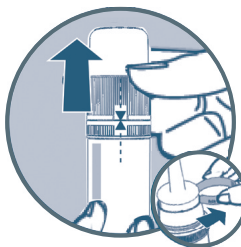

Step 3 - Align the triangles on cap and bottle and pull up the protective cap, then remove the safety clip. Before first use, prime the bottle with several pumps, with bottle upright, until solution is expelled.



Step 4 - Gently insert the bottle tip just into one nostril. Press on the other side of your nose with one finger to close off the other nostril. Aim slightly away from the center of your nose and pump one spray. Repeat in the other nostril. Repeat in each nostril for a total of 4 sprays (2 in each nostril). Sniff gently through the nose while applying but do not inhale deeply. Exhale through your mouth.

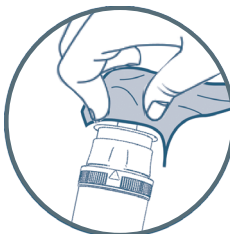

Step 5 - Wipe the sprayer with a clean tissue to maintain hygiene.



ENSURE SPRAYER IS HELD VERTICALLY WHEN APPLYING.